


Sizing guide instructions:

1. Print this sizing guide to scale onto US Letter or A4 sized paper. Ensure you select "Actual Size" and not "Fit" in the printer settings.
2. Place your ruler next to a ruler printed below to ensure they measure the same. If they don't, please contact your healthcare provider.
3. With your scissors, carefully cut along the dotted lines on the sizing guide.
4. Follow the directions on the sizing guide to determine the recommended seal size.

Sizing Tip

Use a mirror or someone at home to assist you with sizing.

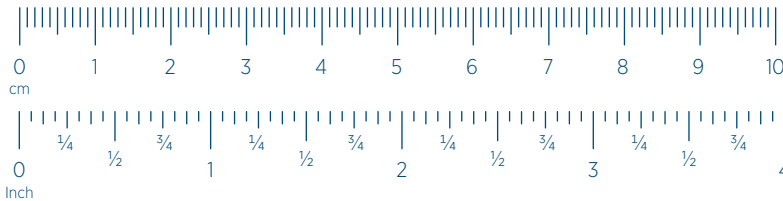

This product may not be right for you. Read the warnings and instructions for use before purchase. Evora Nasal mask may not be available in all markets.

