Clinician Guide

F&P SleepStyle+ Auto
F&P SleepStyle+ CPAP

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LANGUAGES

LANGUAGE	SECTION
English	A
Français canadien (French for Canada)	B

CLINICIAN GUIDE

To be retained by the clinician.

This guide refers to the F&P SleepStyle+ Auto and F&P SleepStyle+ CPAP as the “device”.

The device treats Obstructive Sleep Apnea (OSA) by delivering a flow of continuous positive airway pressure (CPAP) at a level prescribed by the physician to splint open the airway and prevent airway collapse.

Please refer queries relating to the device or accessories to your Fisher & Paykel Healthcare representative.

Note: *This guide is for the clinician only. To prevent patients altering their therapeutic pressure, please ensure they do not have access to this guide.*

Intended use:

The device is for use on adult patients for the treatment of Obstructive Sleep Apnea (OSA).

The device is for use in the home or sleep laboratory.

TABLE OF CONTENTS

1. WARNINGS	A-3
2. GETTING STARTED	A-3
2.1 Device and Accessories.....	A-4
2.2 F&P SleepStyle+ Models and Features.....	A-5
2.3 First Start-up.....	A-5
3. USING THE CLINICIAN MENU	A-6
3.1 How to Enter the Clinician Menu	A-6
3.2 How to Exit the Clinician Menu.....	A-6
3.3 F&P SleepStyle+ Controls for Clinician Menu	A-7
3.4 Clinician Sub-menus	A-8
4. THERAPY OPTIONS	A-11
4.1 Therapy Mode.....	A-11
4.2 Setting the Therapy Pressure.....	A-11
4.2.1 CPAP mode.....	A-11
4.2.2 Auto mode (Auto models only)	A-11
5. COMFORT OPTIONS	A-12
5.1 Humidity.....	A-12
5.2 Auto Start / Stop.....	A-12
5.3 Expiratory Relief.....	A-13
5.4 SensAwake™.....	A-14
6. DEVICE OPTIONS.....	A-15
6.1 Display pressure	A-15
6.2 Display (Patient) AHI.....	A-15
6.3 Time.....	A-16
6.4 Language	A-16
7. SERVICE OPTIONS.....	A-16
7.1 Information.....	A-16
7.2 Fault Log.....	A-16
7.3 Clear Patient Data.....	A-17
7.4 Factory Reset.....	A-17

8. PATIENT SETUP.....	A-18
8.1 Device Setup.....	A-18
8.2 Starting Therapy.....	A-20
8.3 Stopping Therapy.....	A-20
8.4 Setting up the Device with Supplemental Oxygen.....	A-21
8.5 User Controls for the Patient Menu.....	A-22
8.6 Patient Menu Settings.....	A-23
8.6.1 Ramp.....	A-23
8.6.2 Humidity.....	A-23
8.6.3 Expiratory relief.....	A-23
8.6.4 SensAwake.....	A-23
8.6.5 Auto Start / Stop.....	A-23
8.6.6 My Data.....	A-24
8.7 Hidden Data Transfer Menu.....	A-24
8.7.1 Access the hidden data transfer menu.....	A-24
8.7.2 Send my data.....	A-24
8.7.3 Cellular Modem (On/Off).....	A-25
8.7.4 Bluetooth® Technology (On/ Off).....	A-25
8.7.5 Clear all paired devices.....	A-25
8.8 Stand-by Mode.....	A-25
9. SERVICING.....	A-26
10. MULTI-PATIENT USE.....	A-27
10.1 Disassembly for Cleaning and High-Level Disinfection.....	A-27
10.2 Cleaning and High-Level Disinfection of the Device and Accessories.....	A-28
10.2.1 Cleaning.....	A-29
10.2.2 High-level Disinfection.....	A-30
10.3 Reassembly of the Device and Accessories.....	A-31
10.4 Replacing the Air Filter between Patients.....	A-32
10.5 Deleting Therapy Data between Patients.....	A-32
11. THERAPY AND COMPLIANCE DATA.....	A-33
11.1 Modem.....	A-33
11.2 Bluetooth Technology.....	A-33
11.3 F&P InfoUSB™.....	A-33
12. REPLACEMENT PARTS.....	A-34
13. TROUBLESHOOTING.....	A-35
13.1 Device Troubleshooting.....	A-35
13.2 Error Messages on SleepStyle Screen.....	A-37

1. WARNINGS

Refer to the F&P SleepStyle+ Use and Care Guide for contraindications, warnings, cautions, and information essential for safe operation, transport, storage and set up of the device.

Refer to section 10 – *Multi-Patient Use* for cleaning and high-level disinfection of the device.

2. GETTING STARTED





Before using this Clinician Guide, please ensure you have read and understood the F&P SleepStyle+ Use and Care Guide.

You must be familiar with the contraindications, warnings, cautions, symbols, definitions and operating instructions which apply to the device.


2.1 Device And Accessories

- 1 x Carry-bag
- 1 x SleepStyle+ device
- 1 x Breathing tube
- 1 x Power cord
- 1 x F&P SleepStyle+ Use and Care Guide
- 1 x F&P SleepStyle+ Quick Reference Guide
- 1 x Water chamber


- 1 x Chamber seal
- 1 x Outlet seal
- 1 x F&P InfoUSB (already in InfoUSB port)
- 1 x Air filter (already in the device)
- 1 x Spare air filter
- 1 x Spare elbow (for use with a standard breathing tube)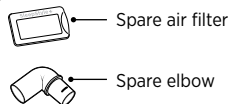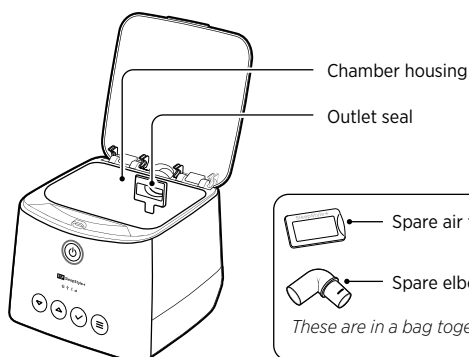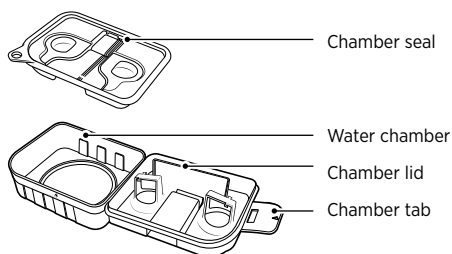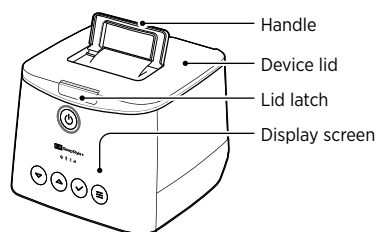

These are in a bag together

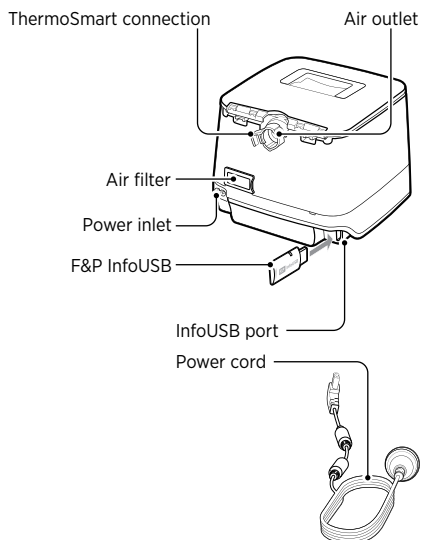ThermoSmart breathing tube **or** standard breathing tube with elbow

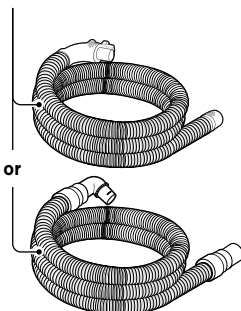2.2 F&P SleepStyle+ Models and Features

Device model	SleepStyle+ Auto	SleepStyle+ CPAP
Australasia	SSPAAA	SSPCAA
North America	SSPAAN	

Performance features	
Fully integrated humidifier	Auto-altitude adjustment
ThermoSmart technology**	Leak compensation
Auto-adjusting pressure†	Efficacy reporting
SensAwake	Compliance reporting
Expiratory relief	F&P InfoUSB
Auto Start / Stop	Bluetooth wireless technology*
Central Apnea Event Detection	Cellular modem*

Compatible with	
F&P InfoSmart™	F&P SleepStyle App and Web‡

* Not available in all models.

** The ThermoSmart Breathing Tube is required to activate ThermoSmart.

† SleepStyle Auto only.

‡ Not available in all countries.

2.3 First Start-up

When the device is powered up for the first time, or after a factory reset, you will be asked to select the language used in the device menus. The default language may vary.

Press **Down** ▾ or **Up** ▴ to scroll through the languages if necessary.

Press **OK** ✓ to confirm your selection.

You will be greeted in the selected language and taken to the device's home screen.

3. USING THE CLINICIAN MENU

3.1 How to Enter the Clinician Menu

1. Press and hold **Start/Stop** 

2. With your other hand, press and hold any one of the four buttons on the bottom row (**Down** , **Up** , **OK** , or **Menu** ) for 3 seconds until "CLINICIAN MENU" appears on the screen.

3.2 How to Exit the Clinician Menu

1. On the main Clinician Menu screen, press **Menu** .


Alternatively, press **Down**  or **Up**  to scroll to "**Exit clinician menu**", located at the top and bottom of the main Clinician Menu screen

2. You will be asked if you would like to "**Exit clinician menu?**".
Press **Down**  or **Up**  to select **Yes** .
3. Press **OK**  to confirm your selection.

Note: The Clinician Menu will default back to the Patient Menu after 15 minutes of no interaction, or if **Start/Stop**  is pressed. Pressing **Start/Stop**  will exit the Clinician Menu and start, or stop, therapy.

3.3 F&P SleepStyle+ Controls for Clinician Menu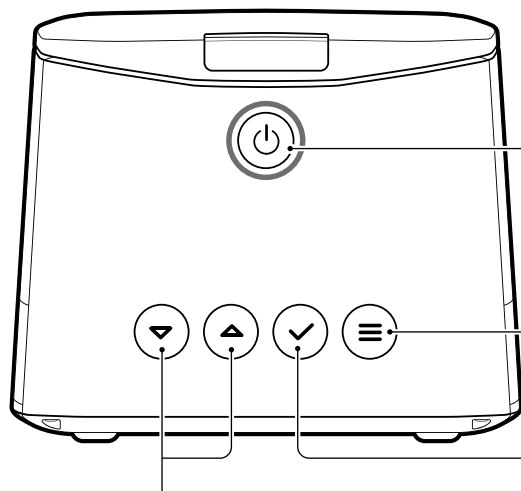

Start/Stop

- Press and hold , along with **Down** ▽, **Up** △, **OK** ✓, or **Menu** ≡, to enter the Clinician Menu.

Menu

- Press to go back one level.
- Press to exit to the Patient Menu from the main Clinician Menu screen.

Down and Up

- Press to navigate the device menu.
- Press to adjust a comfort setting.
- Press to move between options in a setting.

OK

- Press to select a setting.
- Press to make a selection.
- Press to accept an instruction on the display screen.

3.4 Clinician Sub-menus

The Clinician Menu has four sub-menus: Therapy, Comfort, Device, and Service Info.

Below is a guide to the settings in each sub-menu that you can Activate/Deactivate, or, choose to show/ hide information that appears in the Patient Menu. For more detailed information, please refer to the relevant section in this Clinician Guide.

Sub-menu	Settings
Therapy Options Auto Mode 	Therapy Mode Auto: Selects auto-adjusting mode and the minimum and maximum pressure range. CPAP: Selects fixed pressure mode and allows you to set the prescribed pressure.
CPAP Mode 	Auto Mode Min and Max pressure: Sets minimum and maximum pressure. These can also be adjusted independently if Auto Mode has been selected. CPAP Mode Set pressure: Allows you to set the prescribed pressure.
Comfort Options 	Humidity Adjust the level of humidity; the default setting is 5. Auto Start / Stop Activate Auto Start / Stop: Activates or deactivates the Auto Start / Stop feature. If No ☐ is selected, Auto Start / Stop will be Deactivated, you will be taken back to the main Clinician Menu screen, and the Auto Start / Stop Patient Menu item will be removed from the Patient Menu. Allow patient to deactivate: If Yes ☒ is selected, Auto Start / Stop will be visible in the Patient Menu and the patient will be able to turn Auto Start / Stop On or Off. If No ☐ is selected, Auto Start / Stop will not be visible from the Patient Menu so that the patient cannot choose to switch Auto Start / Stop On or Off.

Sub-menu	Settings
Comfort Options 	Expiratory relief Activate Expiratory relief: Activates or Deactivates the expiratory relief feature. If No ☐ is selected, expiratory relief will be Deactivated, you will be taken back to the main Clinician Menu screen, and the “Expiratory Relief” Patient Menu item will be removed from the Patient Menu. Expiratory relief (level): When expiratory relief is Activated, the next automatic menu screen will allow you to adjust the expiratory relief level from 0 – 3 pressure units (cmH₂O). Allow patient to adjust: If Yes ☒ is selected, expiratory relief will be visible in the Patient Menu and the patient will be able to adjust the expiratory relief level from Off (0 shaded circles) to On, and up to 3 (3 shaded circles). If No ☐ is selected, expiratory relief will disappear from the Patient Menu so that the patient cannot choose to adjust the expiratory relief level. Expiratory relief will still run in the background as set up by the clinician. SensAwake Activate SensAwake: Activates or Deactivates the SensAwake feature. If No ☐ is selected, SensAwake will be Deactivated, you will be taken back to the main Clinician Menu screen, and the SensAwake Patient Menu item will be removed from the Patient Menu. SensAwake pressure: When SensAwake is Activated, the next automatic menu screen will allow you to adjust the SensAwake pressure value. Allow patient to deactivate: If Yes ☒ is selected, SensAwake will be visible in the Patient Menu and the patient will be able to turn SensAwake On or Off. Patients cannot adjust the SensAwake pressure. If No ☐ is selected, SensAwake will not be visible from the Patient Menu so that the patient cannot choose to switch SensAwake On or Off. SensAwake will still run in the background as set up by the clinician.

Sub-menu	Settings
Device Options DEVICE Display pressure ▶ Display AHI ▶ Time ▶ Language ▶	Display pressure Option to show or hide pressure whilst therapy is on. Also set the pressure units of the device. Display AHI Option to show or hide AHI when a patient views their therapy data. Time Set the time of the device's internal clock to ensure therapy data is reported accurately (24 hour format). Language Set the language that you would like the device to display.
Service Options SERVICE INFO Information ▶ Fault log ▶ Clear patient data ▶ Factory reset ▶ Exit clinician menu	Information View software version and total runtime hours of the device. Fault log A list of fault codes and the dates on which they occurred since last factory reset. Clear patient data Clears all therapy data from the device and the InfoUSB linked to the device. Factory reset A factory reset removes all data and prescription settings from the device and InfoUSB. It will restart the device with the default factory settings. To avoid data corruption, please do not turn off the device while the factory reset is in progress. After a factory reset, the device will restart as if it is being started for the first time. Refer to section 2.3 – First Start-up.

4. THERAPY OPTIONS

4.1 Therapy Mode

In this menu, you can select the pressure mode.

This can be changed from CPAP (fixed pressure), to Auto (auto-adjusting pressure) in device models with this feature (F&P SleepStyle+ Auto only).

1. Press **Down** ▽ or **Up** △ to scroll to the "Therapy mode" menu.
2. Press **OK** ✓ to select "Therapy mode".

3. Press **Down** ▽ or **Up** △ to change the selection.
4. Press **OK** ✓ to confirm your selection.


Note: Pressing **OK** ✓ will automatically take you to set the therapy pressure. Pressing **Menu** ≡ will take you back to the main Clinician Menu.

4.2 Setting the Therapy Pressure

4.2.1 CPAP mode

This setting allows the prescribed fixed pressure to be set.


The device has a default fixed pressure setting of 10 cmH₂O, which can be adjusted in increments of 0.5 cmH₂O within the range of 4 to 20 cmH₂O.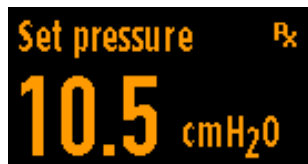

1. Press **Down** ▽ or **Up** △ to change the prescribed pressure.
2. Press **OK** ✓ to confirm and exit to the main Clinician Menu.


4.2.2 Auto mode (Auto models only)

Minimum pressure: Adjust the minimum starting pressure in 0.5 cmH₂O increments from 4 cmH₂O to the maximum pressure setting. The default setting is 4 cmH₂O.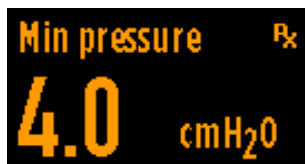

1. Press **Down** ▽ or **Up** △ to change the minimum pressure.
2. Press **OK** ✓ to confirm and move on to maximum pressure.

Maximum pressure: Adjust the maximum pressure in 0.5 cmH₂O increments from the minimum pressure setting to 20 cmH₂O. The default setting is 18 cmH₂O.

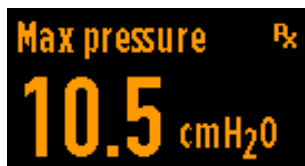1. Press **Down** ▽ or **Up** △ to change the maximum pressure.
2. Press **OK** ✓ to confirm and exit to the main Clinician Menu.

Note: Minimum and maximum pressure can be accessed independently from the main Clinician Menu, under Therapy.

5. COMFORT OPTIONS

5.1 Humidity

The humidity level can be set to 0 (Off) or between 1 and 7 (On). The default humidity setting is 5.

1. Press **Down** ▽ or **Up** △ to scroll to the “Humidity” setting.
2. Press **OK** ✓ to select “Humidity”.

3. Press **Down** ▽ or **Up** △ to change the level of humidity.
4. Press **OK** ✓ to accept the change.

If a ThermoSmart breathing tube is connected to the device, the ThermoSmart icon will appear on the Patient Menu home screen.


5.2 Auto Start / Stop

In this menu, expiratory relief can be Activated or Deactivated.

If Activated, Auto Start / Stop enables the CPAP to automatically start therapy whenever the patient breathes into the mask, and stop therapy after the mask is removed for a short period of time.

You can also select whether the patient is able to turn Auto Start / Stop On or Off.

1. Press **Down** ▽ or **Up** △ to scroll to the “Auto Start / Stop” setting.
2. Press **OK** ✓ to select “Auto Start / Stop”.

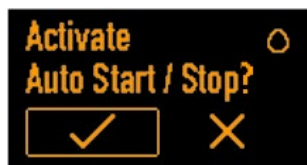3. You will be asked if you would like to “Activate Auto Start / Stop”. Press **Down** ▽ or **Up** △ to change the setting.

4. Press **OK** ✓ to accept the change.

If you have Activated Auto Start / Stop:

You will be asked if you would like to “Allow patient to deactivate?”. This allows you to set whether the patient can turn Auto Start / Stop On or Off.

5. Press **Down** ▽ or **Up** △ to change the setting. **Yes** ✓ will allow the patient to turn Auto Start / Stop On or Off. **No** ✗ will remove Auto Start / Stop from the Patient Menu. Auto Start / Stop will still run in the background as set up by the clinician.
6. Press **OK** ✓ to accept the change and exit to the main Clinician Menu.

5.3 Expiratory Relief

In this menu, expiratory relief can be Activated or Deactivated.

If Activated, the expiratory relief level can be set to Off (3 transparent circles), Low (1), Medium (2), or High (3). The default setting is Activated and the default expiratory relief level is Low (1).

You can also select whether the patient is able to adjust expiratory relief.

1. Press **Down** ▽ or **Up** △ to scroll to the “Expiratory relief” setting.
2. Press **OK** ✓ to select “Expiratory relief”.

3. You will be asked if you would like to “Activate Expiratory relief”. Press **Down** ▽ or **Up** △ to change the setting.
4. Press **OK** ✓ to accept the change.

If you have activated expiratory relief:

5. Press **Down** ▽ or **Up** △ to change the expiratory relief level.
6. Press **OK** ✓ to confirm and exit to the main Clinician Menu.

You will be asked if you would like to “Allow patient to adjust?”. This allows you to set whether the patient can turn expiratory relief On or Off and adjust the expiratory relief level:

7. Press **Down** ▽ or **Up** △ to change the setting.

Yes ✓ will allow the patient to turn expiratory relief On or Off and adjust the expiratory relief level. **No** ✗ will remove expiratory relief from the Patient Menu.

Expiratory relief will still run in the background as set up by the clinician.

8. Press **OK** ✓ to accept the change and exit to the main Clinician Menu.

5.4 SensAwake™

In this menu, SensAwake can be Activated or Deactivated.


If Activated, the SensAwake pressure can be set between 4 and 20 cmH₂O, in 0.5 cmH₂O increments. The default setting is Activated and the default SensAwake pressure is 4 cmH₂O.

You can also select whether the patient is able to turn SensAwake On or Off. Patients cannot adjust the SensAwake pressure.

1. Press **Down** ▽ or **Up** △ to scroll to the “SensAwake” setting.
2. Press **OK** ✓ to select “SensAwake”.

3. You will be asked if you would like to “Activate SensAwake”. Press **Down** ▽ or **Up** △ to change the setting.
4. Press **OK** ✓ to accept the change.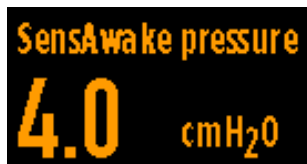

If you have Activated SensAwake:

5. Press **Down** ▽ or **Up** △ to change the SensAwake pressure.
6. Press **OK** ✓ confirm your selection.

You will be asked if you would like to “Allow patient to deactivate?”. This allows you to set whether the patient can turn SensAwake On or Off.

7. Press **Down** ▽ or **Up** △ to change the setting.
Yes ☒ will allow the patient to turn SensAwake On or Off. **No** ☐ will remove SensAwake from the Patient Menu. SensAwake will still run in the background as set up by the clinician.
8. Press **OK** ✓ to accept the change and exit to the main Clinician Menu.

Note: In Auto mode, SensAwake pressure cannot be set above the minimum pressure. In CPAP mode, SensAwake pressure cannot be set above the prescribed fixed pressure. This is the pressure the patient will receive when they start therapy until a sleep-disordered breathing event occurs.

6. DEVICE OPTIONS

6.1 Display pressure

Display pressure allows you to show or hide the pressure level during therapy. The default setting is pressure hidden during therapy. This option also allows you to set the pressure units of the device in cmH₂O or hPa.

1. Press **Down** ▾ or **Up** ▴ to scroll to the “Display pressure” setting.
2. Press **OK** ✓ to select “Display pressure”.

3. Press **Down** ▾ or **Up** ▴ to change the setting. If you select **No** [X], the patient will not see their pressure during therapy. If you select **Yes** [✓], the patient will see their pressure during therapy.
4. Press **OK** ✓ to accept the change. You will now see the pressure unit selection screen.

5. Press **Down** ▾ or **Up** ▴ to change the setting. The units you select will be displayed during therapy (if this has been enabled in the previous screen) and when selecting therapy pressure.
6. Press **OK** ✓ to confirm your selection.

6.2 Display (Patient) AHI

Display AHI allows you to show or hide AHI when a patient views their therapy data. The default setting is AHI shown in the Patient Menu.

1. Press **Down** ▾ or **Up** ▴ to scroll to the “Display AHI” setting.
2. Press **OK** ✓ to select “Display AHI”.

3. Press **Down** ▾ or **Up** ▴ to change the setting.

If you select **No** [X], the patient will not see their AHI when viewing “My Data” in the Patient Menu or in any of the data screens displayed when they stop therapy. If you select **Yes** [✓], the patient will see their AHI when viewing “My Data” and in the data screens displayed when they stop therapy.

4. Press **OK** ✓ to accept the change and exit to the main Clinician Menu.

Note: The device-reported AHI is an estimate based on flow, and therefore should not be used for diagnosis of OSA or in isolation for titration or clinical management.

6.3 Time

To ensure accurate therapy data reporting, the time on the device's internal clock needs to be set.

The time must be set in 24 hour format.

To set the device time:

1. Press **Down** ▽ or **Up** △ to scroll to "Time" and press **OK** ✓ to select.

2. Press **Down** ▽ or **Up** △ to select the correct Hour and press **OK** ✓. Press and hold **Down** ▽ or **Up** △ to fast-scroll.

3. Press **Down** ▽ or **Up** △ to select the correct Minutes and press **OK** ✓. Press and hold **Down** ▽ or **Up** △ to fast-scroll.

6.4 Language

The language screen allows you to select the language that appears on the device for the Patient and Clinician Menus.

4. Press **Down** ▽ or **Up** △ to scroll to "Language".
5. Press **OK** ✓ to select.


6. Press **Down** ▽ or **Up** △ to select the language.
7. Press **OK** ✓ to accept the change and exit to the main Clinician Menu.

7. SERVICE OPTIONS

7.1 Information

The Information screen shows the software version the device is running and the total number of runtime hours of the device.

1. Press **Down** ▽ or **Up** △ to scroll to "Information".
2. Press **OK** ✓ to select.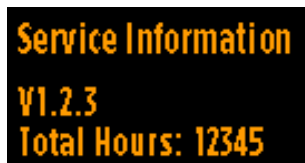


3. Press **OK** ✓ or **Menu** ≡ to exit to the main Clinician Menu.

7.2 Fault Log

This is a list of fault codes and the dates on which they occurred since the last factory reset.

1. Press **Down** ▽ or **Up** △ to scroll to "Fault log".
2. Press **OK** ✓ to select.

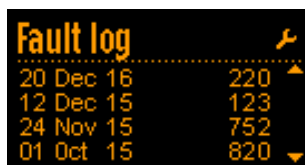3. Press **Down** ▽ or **Up** △ to scroll through the list of recorded faults.
4. Press **OK** ✓ or **Menu** ≡ to exit to the main Clinician Menu.

7.3 Clear Patient Data

“Clear patient data” allows you to delete all of the therapy data from the device and the associated InfoUSB.

1. Make sure the InfoUSB is plugged into the device.
2. Press **Down** ▽ or **Up** △ to scroll to “Clear patient data”.
3. Press **OK** ✓ to select.

4. You will be asked “Are you sure?”. If you select **No** [X], you will exit to the main Clinician Menu and therapy data will not be cleared. If you select **Yes** [✓], a clearing sequence will start.

5. Press **OK** ✓ to exit to the main Clinician Menu once the clearing sequence is complete. Alternatively, the device will automatically exit after 3 seconds.

7.4 Factory Reset

A factory reset will delete all data and prescription settings from the device and the InfoUSB. Please do not turn off the device while the factory reset is in progress. The device will restart with the default factory settings.

1. Make sure the InfoUSB is plugged into the device if you would like it to be cleared.
2. Press **Down** ▽ or **Up** △ to scroll to “Factory reset”.
3. Press **OK** ✓ to select.

4. You will be asked “Are you sure?”. If you select **No** [X], you will be taken to the main Clinician Menu and data will not be cleared. If you select **Yes** [✓], a clearing sequence will start.

5. Once the clearing sequence is complete, the device will restart and you will be taken through the First-Start up sequence. See section 2.3 – First Start-Up.

Note: If the InfoUSB is not in the device at the time of the factory reset, it will not be cleared. The next time it is inserted into the device, compliance data on the InfoUSB will be overwritten. If there is data from other devices on the InfoUSB, that data will not be deleted at the time of the factory reset. You can manually clear all data from the InfoUSB using a computer.


8. PATIENT SETUP

8.1 Device Setup

Below is a guide for taking a patient through the device setup. You may wish to refer the patient to the F&P SleepStyle+ Quick Reference Guide.

Note: Please ensure the patient has read and understood all Warnings, Cautions, and setup instructions in the F&P SleepStyle+ Use and Care Guide.



1. Place the device below head height on a stable and level surface (like a bedside table).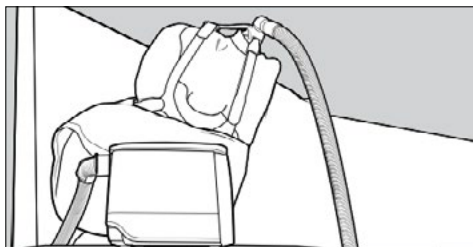

Warnings

To avoid injury, choking, or inhalation of a foreign object, and to prevent water from entering the breathing tube:
Do not place the device above head height.

2. Connect the power cord and ThermoSmart breathing tube (or the standard breathing tube and elbow).
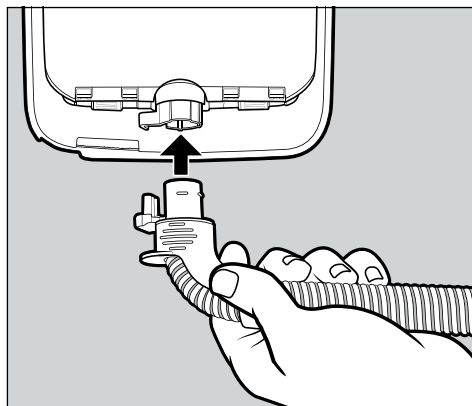
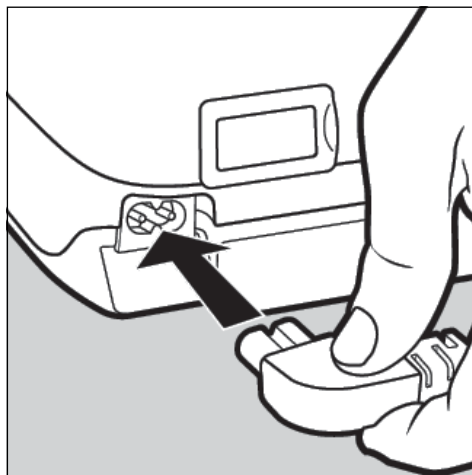


Warnings


To avoid electric shock:
Do not use if the device, power cord, or accessories are damaged, deformed or cracked.

To avoid choking, or inhalation of a foreign object:
Ensure the breathing tube and power cord, including any extension cords, are correctly positioned so they will not become entangled with the body or furniture during sleep.

3. Open the device lid and remove the water chamber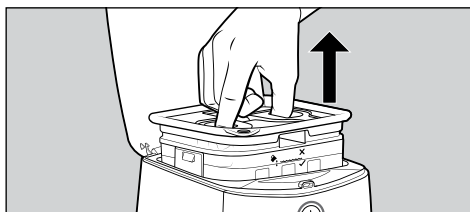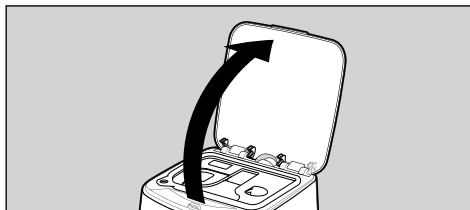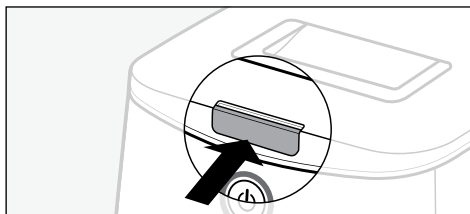

4. Fold back the chamber seal using the corner tab. Fill the water chamber up to the maximum water-level line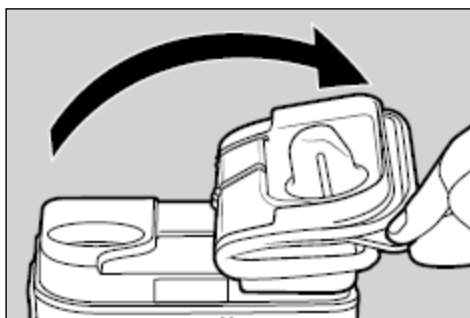

Warnings

To avoid burns:

Do not fill the water chamber with hot water as this may lead to airway burns

Cautions


To prevent water damage to the device:

- Do not use if the water chamber is damaged.
- Do not fill the chamber housing with water. Only place water in the water chamber.
- Do not fill the water chamber above the maximum water-level line.
- Replace water before each use.
- Do not fill the water chamber while it is in the device.
- Do not use the device with an empty water chamber unless the humidity level is set to 0.
- Do not add aromatic-based or scented oils to the water chamber as these oils can cause damage to the device.

General:

Use distilled water to reduce residue build-up on the chamber base. This will extend the life of the water chamber.


5. Unfold the chamber seal and push eplace.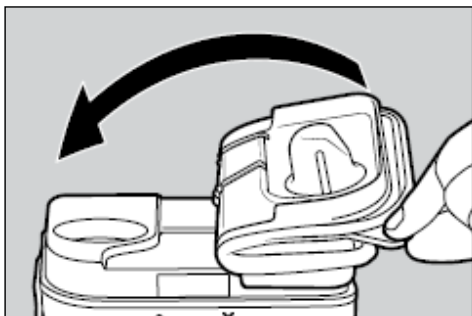

⚠ Cautions

To prevent water damage to the device:
Do not use the device without the chamber seal fitted to the water chamber.

6. Place the water chamber back into the device and close the device lid.

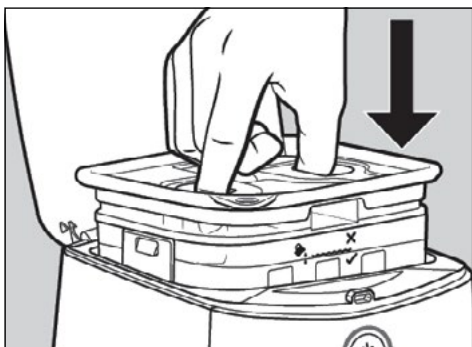8.2 Starting Therapy

1. Ask the patient to fit their mask.
2. Press **Start/Stop**  to start therapy.
3. The screen below will appear:

Note: If Auto Start/Stop is activated, the device will automatically start therapy whenever you breathe into your mask.

4. If using supplemental oxygen, turn on the oxygen supply once therapy has started.

8.3 Stopping Therapy

1. Ask the patient to remove their mask.
2. If using supplemental oxygen, turn off the oxygen before turning off therapy on the device.
3. Press **Start/Stop**  to stop therapy.

The device screen below will appear:

Note: If Auto Start / Stop is activated, the device will automatically stop therapy after the mask is removed for a short period of time.

4. The screen will then scroll through the patient's therapy data from the session that has just finished (if the session has lasted 6 minutes or longer).

Therapy Hours: Number of hours the patient used their device.

Mask Leak: If the patient's mask leak was "Normal" or "High".

AHI: Average number of apneas or hypopneas the patient had per hour.


Note: If you would like to bypass this automated scrolling, press **Menu**  to scroll through the data screens manually.

8.4 Setting up the Device with Supplemental Oxygen

Supplemental oxygen can be administered at the mask end of the breathing tube up to a maximum flow rate of 5 litres per minute as follows:

1. Set up the device by following steps 1 – 6 of section 8.1: *Device Setup*.
2. Connect the Oxygen/Pressure Port Connector 900HC452 between the mask swivel/inlet and the breathing tube outlet.
3. Connect the oxygen tube to the Oxygen/Pressure Port Connector 900HC452.

Refer to diagram below:

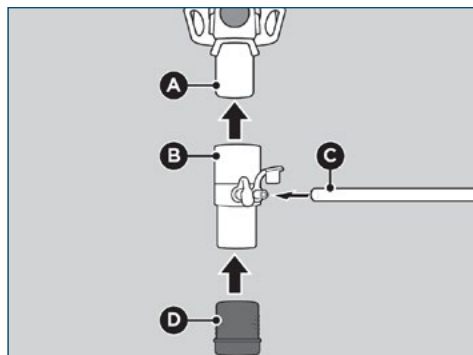- A. Mask Swivel/Inlet
- B. Oxygen/Pressure Port Connector 900HC452
- C. Oxygen Tube
- D. Breathing Tube Outlet

Refer to Oxygen/Pressure Port Connector 900HC452 User Instructions for more information including cleaning.

Warnings

To avoid risk of fire with supplemental oxygen:


- Turn on therapy on the device prior to turning on the oxygen supply. Turn off oxygen supply before turning off therapy on the device. If oxygen is left on, it can accumulate inside the device and create a fire risk.
- Ensure adequate ventilation is provided around the device.
- Remove any source of ignition, such as cigarettes, an open flame, or materials which burn or ignite easily at high oxygen concentration.
- Do not smoke when using supplemental oxygen.
- Keep oxygen regulators, cylinder valves, tubing, connections and all other oxygen equipment away from oil, grease or greasy substances. Spontaneous and violent ignition may occur if these substances come into contact with oxygen under pressure.

Notes:

- Ensure there is no obstruction downstream of the Oxygen/Pressure Port Connector 900HC452 as this can affect the delivered oxygen concentration.
- At a fixed flow rate of supplemental oxygen, the inhaled oxygen concentration will vary, depending on the pressure settings, the patient's breathing pattern, mask selection and leak rate
- It is recommended that the oxygen concentration is measured at the point of delivery to the patient.

8.5 User Controls for the Patient Menu

The Patient Menu has different controls to the Clinician Menu. When in the Patient Menu, please use the controls as outlined below.

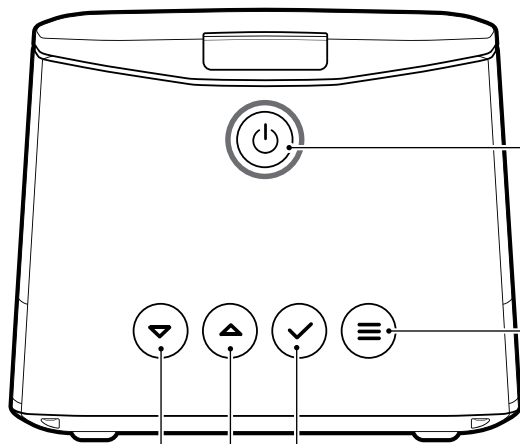Start/Stop

- Press and hold , along with **Down** ▽, **Up** △, **OK** ✓, or **Menu** ≡, to enter the Clinician Menu.

Menu

- Press to go back one level.
- Press to exit to the Patient Menu from the main Clinician Menu Screen.

Down and Up

- Press to navigate the device menu.
- Press to adjust a comfort setting.
- Press to move between options in a setting.

OK

- Press to select a setting.
- Press to make a selection.
- Press to accept an instruction on the display screen.

8.6 Patient Menu Settings

The Patient Menu refers to all the settings that the patient can view to access and make adjustments to their therapy. Please refer to the F&P SleepStyle+ Use and Care Guide for more details.

From the home screen, or during therapy, press **Menu**  to access the device menu.

Note: For more information on the Patient Menu home screen, including the four indicator icons, refer to section 3.1 – Screen icons in the F&P SleepStyle+ Use and Care Guide.

8.6.1 Ramp

Ramp works by gradually increasing the air pressure to the prescribed pressure over a 20-minute period. Ramp can be activated in both CPAP and Auto mode.

To start ramp, press and hold **Start/Stop**  until **Ramping**  appears on the screen.

Ramp will increase to the prescribed pressure over a 20-minute period, starting from one third of the prescribed pressure, if the device is in CPAP mode, or minimum pressure, if the device is in Auto mode.

Note: If SensAwake is on, then Ramp is not required if the minimum SensAwake pressure is comfortable for the patient.

8.6.2 Humidity

The patient can set the humidity level to 0 (Off) or between 1 and 7 (On).

8.6.3 Expiratory relief

Expiratory relief is available with all models, and the following settings are available:

Off, low (1), medium (2), or high (3).

8.6.4 SensAwake


SensAwake is available with all models, and the patient can turn SensAwake On or Off.

Note: SensAwake is defaulted to Off in CPAP mode.

8.6.5 Auto Start / Stop

Auto Start / Stop is available with all models, and the patient can turn Auto Start / Stop On or Off.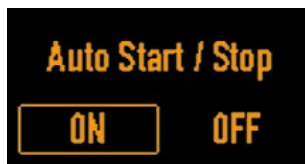

Note: Auto Start / Stop is defaulted to Off.

8.6.6 My Data

This setting allows the patient to view their basic therapy data (daily, weekly, or monthly) on the device, including:

- Therapy Hours – Last night
- Therapy Hours – Last 7 days
- Therapy Hours – Last 30 days
- Mask Leak – Last night
- Mask Leak – Last 7 days
- Mask Leak – Last 30 days
- AHI – Last night
- AHI – Last 7 days
- AHI – Last 30 days

The AHI screens can be hidden from the Patient Menu. In the Clinician Menu, go to the “Device” sub-menu, scroll to “Display AHI” and then select **No** ☒.

8.7 Hidden Data Transfer Menu

This hidden menu is intended for troubleshooting over the phone, between the clinician and the patient. The following are instructions to provide to the patient to guide them in accessing and navigating this hidden menu.

8.7.1 Access the hidden data transfer menu

From the home screen, or during therapy, hold **Menu** for 5 seconds.

8.7.2 Send my data*

This option allows the patient to manually upload their therapy and compliance data to F&P InfoSmart so you can view the data, instead of waiting for the next scheduled upload, or if an issue has occurred during a scheduled upload.

To manually upload the patient's therapy data to F&P InfoSmart, ask the patient to:

1. Press **Down** or **Up** to change the selection to **Yes** ☒. The device will begin the manual upload of the patient's therapy data. The following screens will appear in sequence as the device searches for a signal and transmits the data (this process may take a few minutes):

Note: If the modem *has* been turned off and the user selects **Yes** ☒, the modem will automatically activate for data transfer.

2. If the data has uploaded successfully, the following screen will appear:

If the data has not uploaded successfully, the following screen will appear instead:

Instruct the patient to attempt a manual upload again. If the error persists, record the number on the screen and contact your Fisher & Paykel Healthcare representative.

3. Once the upload is complete, press **OK** or **Menu** to acknowledge and clear the message and return to the “Send my data” menu.
4. Press **Menu** to scroll to the next screen in the hidden menu. To exit the hidden menu, continue pressing **Menu** until the home screen is reached.

8.7.3 Cellular Modem (On/Off)*

This option allows the patient to turn the cellular modem On or Off.

To turn the cellular modem On or Off, ask the patient to:

1. Press **Menu**  until they reach the "Cellular Modem" setting.

2. Press **Down**  or **Up**  to change the selection. The selected option will flash once to indicate the selection has been saved.
3. Press **Menu**  to scroll to the next screen in the hidden menu. To exit the hidden menu, continue pressing **Menu**  until the home screen is reached.

Note: If turned Off, the modem will automatically turn back On after 3 days.

* Only available in models with a cellular modem.

8.7.4 Bluetooth® Technology* (On/ Off)

This option allows the patient to turn Bluetooth wireless technology On or Off.

To turn Bluetooth On or Off, ask the patient to:

1. Press **Menu**  until they reach the "Bluetooth" setting.

2. Press **Down**  or **Up**  to change the selection. The selected option will flash once to indicate the selection has been saved.
3. Press **Menu**  to scroll to the next screen in the hidden menu. To exit the hidden menu, continue pressing **Menu**  until the home screen is reached.

8.7.5 Clear all paired devices

If the Bluetooth pairing fails, then clearing all paired devices will allow the patient to restart the pairing process. The patient may also wish to do this should

*Not available in all models.

they choose to forget the F&P SleepStyle+ device from their mobile device.

Clearing all paired devices should only be done as a last resort.

This feature is also useful if you, or the patient, would like to replace the mobile device that the device is currently paired to. The device can pair with multiple mobile devices; however, only one mobile device with the app can see the patient's therapy data.

Note: This option is only available if Bluetooth is On.

To clear all paired devices, ask the patient to:

1. Press **Menu**  until they reach the "Clear all paired devices?" screen.

2. Press **Down**  or **Up**  to change the selection to **Yes** .
3. Press **OK**  to clear all paired devices. Wait for the following screen to appear:

4. Press **OK**  or **Menu**  to acknowledge and clear the message.
5. Press **Menu**  to return to the Patient Menu home screen.

8.8 Stand-by Mode

The device will enter stand-by mode after 30 seconds if no button has been pressed on the device.

The display screen light will dim but will still be visible to show that the device is still powered on. Press **Down**  or **Up** , **OK** , or **Menu**  to wake up the device.

9. SERVICING

Warnings

General:

This device is not repairable and does not contain any repairable parts. Please refer queries relating to the device or accessories to your healthcare provider.

This device contains a non-replaceable lithium coin cell battery that is hazardous. Keep away from children. Seek immediate medical attention if it is suspected the button cell battery has been swallowed or placed inside any part of the body, as severe or fatal injuries may occur within 2 hours.

The device does not require preventive maintenance.

10. MULTI-PATIENT USE

High-level disinfection cleaning is required when the device is intended for multiple-patient use.

10.1 Disassembly for Cleaning and High-Level Disinfection

DEVICE

Disconnect the device from the mains power supply and remove the power cord from the rear of the device.


Warnings


To avoid electric shock:
Do not pull on the power cord as it may become damaged.

BREATHING TUBE

1. Hold the plastic connector and gently pull it away from the device.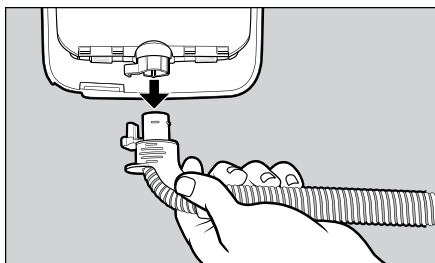


ThermoSmart breathing tube

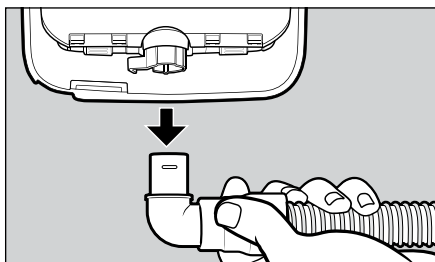Standard breathing tube


2. Hold both the mask end of the tube and the mask swivel and gently pull them apart.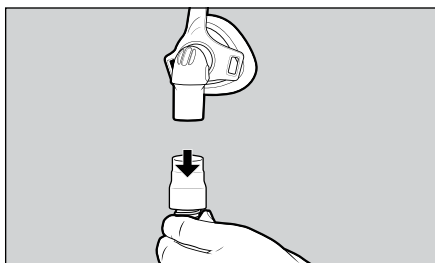


3. Disconnect the elbow from the standard breathing tube.

WATER CHAMBER AND CHAMBER SEAL


1. Press the lid latch and open the device lid.
2. Take the water chamber out of the device.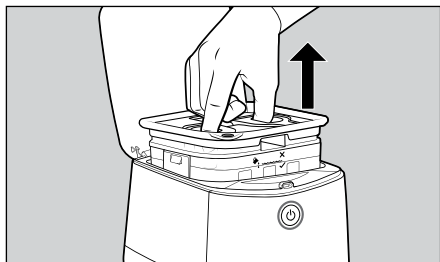


3. Remove the chamber seal from the top of the water chamber and put aside.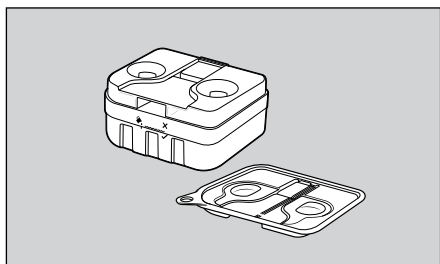

4. Lift the chamber tab and lift the chamber lid to open.

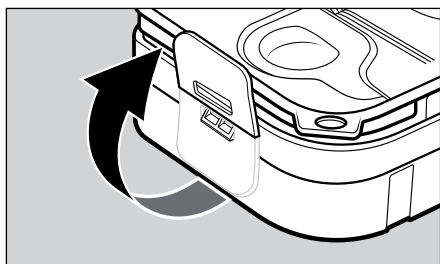**OUTLET SEAL**

1. Grip the outlet seal tab.
2. Gently pull the outlet seal out of the device.

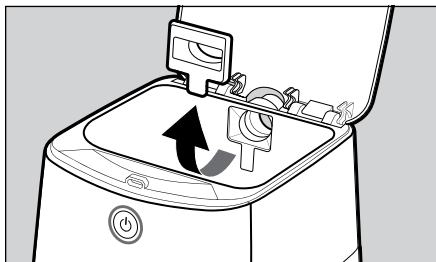10.2 Cleaning and High-Level Disinfection of the Device and Accessories

The device humidified air path including the chamber, chamber seal, outlet seal and breathing tube can become contaminated with body fluids or by contaminants carried by expired breathing gases. To maintain optimal therapy and presentation of the device and accessories, regular cleaning is recommended. Refer to the cleaning instructions in section 6 - *Caring for Your Device* of the F&P SleepStyle+ Use and Care Guide.

For reprocessing between patients, follow the cleaning and then the high-level disinfection instructions below.

**Warnings**

To avoid electric shock:

- Do not use bleach, alcohol, or cleaners with citrus or other natural oils. These substances may degrade the device and accessories.
- Do not immerse the device in water or any other liquid.

To avoid incorrect therapy:

Only clean the device and accessories according to the cleaning instructions below.

To avoid adverse effects:

Do not use ozone to clean this device. Residual ozone may result in adverse respiratory effects such as irritation of the nose, throat and lungs.

Cautions

General:

Replace the device and accessories if there is any sign of cracking, deformation, discoloration or leaking. It is recommended that you inspect the device, breathing tube, water chamber, chamber seal, outlet seal, air filter and elbow, on a regular basis after cleaning. See section 12 – *Replacement Parts*.

Do not use ozone to clean the device as it can degrade the device and its accessories.

10.2.1 CLEANING

DEVICE

1. Wipe the exterior of the device and the chamber housing with a clean, damp (not dripping) cloth and warm soapy water using a mild dishwashing detergent.
2. Leave to dry out of direct sunlight or heat.

WATER CHAMBER, CHAMBER SEAL, BREATHING TUBE, OUTLET SEAL AND ELBOW

1. Hand-wash these accessories in a tub of warm, soapy water with a mild dishwashing detergent. Ensure all visible soil is removed.
2. Rinse thoroughly in a tub of clean water for 30 seconds. Ensure that all soap residue has been removed.
3. Repeat the rinsing process again, using clean water.
4. Hang the breathing tube, with both ends pointing to the floor, to dry away from direct sunlight or heat.
5. Leave the water chamber, chamber seal, outlet seal and elbow to dry out of direct sunlight or heat.

Note: *If dirt remains inside the breathing tube after rinsing, use a soft, non-metallic brush, to remove it. Rinse the tube again. If the dirt cannot be removed, the breathing tube should be replaced. The elbow needs to be cleaned separately from the standard breathing tube.*

10.2.2 HIGH-LEVEL DISINFECTION

Note: High-level disinfection may be repeated 20 times per part, then they should be replaced. The air filter must be replaced between patients. Refer to section 10.4 – Replacing the Air Filter between Patients. The device lid does not require high-level disinfection.

	Thermal Disinfection		Chemical Disinfection		
	75 °C (167 °F) for 30 minutes	90 °C (194 °F) for 1 minute	CIDEX™ OPA	Anioxide™ 1000	gigasept™ FF
Standard breathing tube	✔	⚠ No	✔	⚠ No	⚠ No
ThermoSmart breathing tube	⚠ No	⚠ No	✔	✔	✔
Elbow	✔	✔	✔	✔	✔
Water chamber	✔	✔	✔	✔	✔
Chamber seal	✔	✔	✔	✔	✔
Outlet seal	✔	✔	✔	✔	✔

THERMAL DISINFECTION

 Warnings

To avoid incorrect therapy:
Do not clean or disinfect the ThermoSmart breathing tube with hot water. This may cause deformation of the tube and reduce therapeutic pressure.

1. Immerse the disassembled parts in a water bath. Ensure that no air bubbles are trapped inside the standard breathing tube.
2. Increase the water bath temperature to 75 °C (167 °F).
3. Soak for 30 minutes.
4. Air-dry out of direct sunlight or heat.

Note: Higher temperatures may damage the standard breathing tube. The elbow, water chamber, chamber seal and outlet seal can be thermally disinfected separately from the standard breathing tube in a water bath at 90 °C (194 °F) for 1 minute.


CHEMICAL DISINFECTION

1. Immerse the disassembled parts in a cleaning solution using CIDEX OPA, Anioxide 1000, or gigasept FF. Follow the manufacturer’s instructions for concentration, temperature, and time.
2. Ensure that no air bubbles are trapped inside the breathing tube. The ThermoSmart breathing tube is buoyant so will need to be submerged (for example, in a metal tray with a mesh top).
3. Thoroughly rinse each part in a tub of water by immersing completely for a minimum of 1 minute.
4. Repeat the rinsing process again.
5. Air-dry out of direct sunlight or heat.

10.3 Reassembly of the Device and Accessories


OUTLET SEAL

Hold the tab on the outlet seal and push it into the chamber housing inlet. Ensure the tab is sitting flat against the wall of the chamber housing.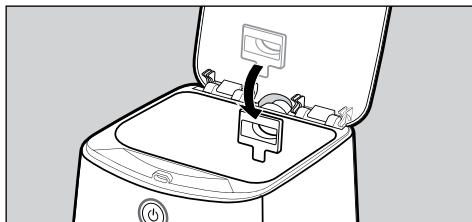


WATER CHAMBER AND CHAMBER SEAL


1. Close the chamber lid. Press the chamber tab down until it clicks into place.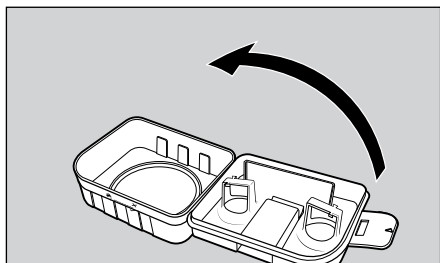

2. Secure the chamber seal back onto the water chamber by pushing down in the finger holds. Ensure it is sitting flat and seals the holes on the chamber lid.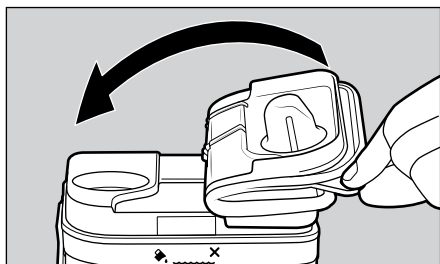

3. Place the water chamber back into the device.
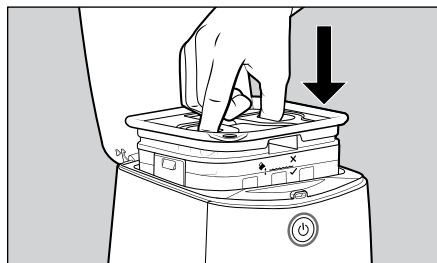


BREATHING TUBE

1. Hold the elbow end of the breathing tube and push it into the air outlet of the device.
Note: If you have a ThermoSmart breathing tube, make sure the electrical connectors on the elbow click into position with the ThermoSmart connection.
2. Holding the mask and the other end of the breathing tube, push the mask swivel firmly into the breathing tube.
3. Reconnect the elbow to the standard breathing tube.

DEVICE

Reconnect the power cord to the rear of the device and reconnect the power cord to the mains power supply.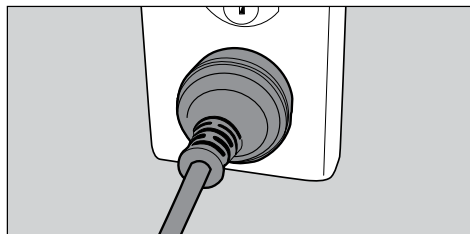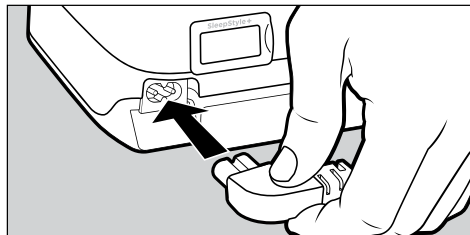


10.4 Replacing the Air Filter between Patients

The air filter is located at the rear of the device and ensures the air intake into the device is filtered to remove particles and dust. The air filter is not washable and must be replaced between patients. Replace the air filter (900SSP113) at least once every 3 months, and the ultra-fine filter (900SSP112) once every month, or more frequently if it becomes blocked with dirt or dust. To replace the air filter, please follow the instructions below.


Warnings

To avoid choking, or inhalation of a foreign object: Do not use the device without the recommended air filter fitted. The filter will reduce dust or particles entering the device and breathing tube.


1. To remove, grip the device and lever the housing out using your finger.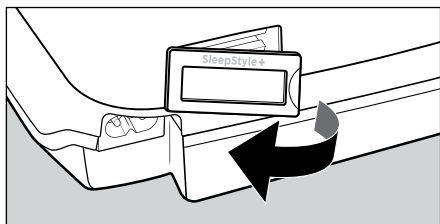


2. If there is an ultra-fine filter in the air filter housing, remove this first by gripping the air filter housing and levering the air filter out.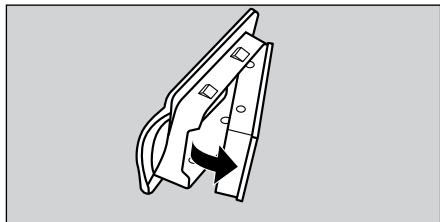

3. If applicable, place an unused, ultra-fine filter into the back of the air filter housing.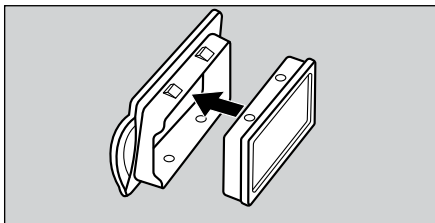

4. Insert the air filter back into the device.

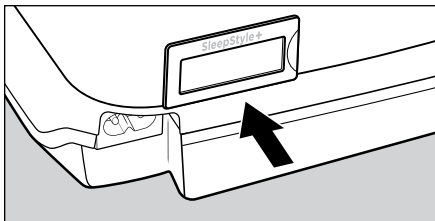10.5 Deleting Therapy Data between Patients

For accurate compliance reports, data must be cleared from the device memory, as well as the InfoUSB, before use by another patient. Follow the instructions in section 7.3 – *Clear Patient Data* to delete therapy data from the device's memory.

11. THERAPY AND COMPLIANCE DATA

The retrieval of therapy and compliance data is available in a range of solutions to suit different patient needs.

11.1 Modem

If the device has a cellular modem, therapy and compliance data will automatically upload from the patient's device to your F&P InfoSmart account.

If data has not been uploaded, you can contact the patient and ask them to manually send the data from their device. Refer to section 8.7.2 - *Send my data* to take the patient through this process.

11.2 Bluetooth Technology*

The Bluetooth wireless technology feature allows patients to pair the device with a mobile device so they can use the F&P SleepStyle App**. The app allows them to view their CPAP therapy progress, set goals, and identify and troubleshoot any issues. You can also view their therapy data when it uploads via the app, if their mobile device has connected to Wi-Fi or a mobile data network.

To use the F&P SleepStyle App via Bluetooth, the patient will need:

- F&P SleepStyle+ with Bluetooth
 - F&P SleepStyle App installed on iPhone and connected to Wi-Fi or a mobile data network. The app is available on the Apple App Store.
- Or
- F&P SleepStyle App installed on any leading smartphone with Android connected to Wi-Fi or a mobile data network. The app is available on the Google Play Store.

Once the app is installed, the patient will need to open the app and follow the on-screen instructions to pair the device with their mobile device via Bluetooth, and then link to their F&P SleepStyle account so they can view their therapy data.

11.3 F&P InfoUSB™

The InfoUSB (stored in the InfoUSB port) records therapy data from the internal memory of the device, and this data can be transferred to a report format using InfoSmart Web* or the InfoSmart application. The following data is recorded in InfoSmart reports:

- 7 days' detailed efficacy data (including AHI, pressure and leak)
- Customizable summary data of up to 12 months
- Cumulative summary data from first use
- Summary report of up to 12 months of data.

With the InfoUSB and SleepStyle Web*, or the InfoUSB application, patients will have the ability to send their therapy and compliance data, as well as receive remote prescription changes via the internet. InfoSmart Web has built-in customizable automated telephone, email, and text messages that can be set to remind the patient to complete the data transfer in a timely manner.

Cautions

General:

Only use the InfoUSB with the device. Use of any other USB drives may cause data corruption. Do not attempt to change the directories or view the data without software distributed or designed for use with the device.

The InfoUSB may become full if used to store other files. This may result in the report not downloading from the InfoUSB or the InfoSmart report having missing data. Refer to section 13.1 - *Device Troubleshooting* if this problem occurs.

**Not available in all models.*

***Not available in all countries.*

12. REPLACEMENT PARTS

Below is a list of replacement parts that are available.

Product code	Description
900SPS100	Water chamber
900SPS101	Chamber seal
900SSP113	Air filter (single)*
900SSP114	Air filters (2-pack)*
900SSP112	Ultra-fine filter**
900SPS120	ThermoSmart breathing tube***
900SPS121	Standard breathing tube with elbow***
900SPS122	Elbow (for use with a standard breathing tube)
900SSP143	Device lid
900SPS141	Outlet seal
900SPS142	Carry bag
900SPS161	Australasian power cord
900SPS160	North American power cord
900SW101	F&P InfoUSB
900HC452	Oxygen/Pressure Port Connector

**The air filter does not include the ultra-fine filter.*

***The ultra-fine filter is an optional accessory that can be purchased separately from your healthcare provider.
900SSP112 contains an air filter with 3 ultra-fine filters*

**** Applied Parts – to fit 22 mm (0.86 in.) Conical Connector.*

 Warnings

To avoid injury:

Do not use breathing tubes, parts, and accessories that are not distributed for use with this device or recommended by Fisher & Paykel Healthcare.

General:

Do not use accessories or power cables which are not provided, or recommended, by Fisher & Paykel Healthcare. This could result in increased electromagnetic emissions or decreased electromagnetic immunity.

13. TROUBLESHOOTING

If you feel that the device is not operating correctly, please refer to the following suggestions. If the problem continues, please contact your Fisher & Paykel Healthcare representative.

Warnings

To avoid electric shock:

- Do not modify the device or accessories.
- Do not take apart the device. Taking the device apart, for example by unscrewing the underside of the device, will damage pressure seals and electrical components.

13.1 Device Troubleshooting

Problem	Solution
There is no visible display or power.	The power cord may not be inserted fully into the device or the plug may not be correctly connected to the power supply. Solution: Ensure the power cord plug is fully inserted into the device and the mains plug is connected to the power supply.
Therapy won't start, but there is a display on the screen.	There may be water in the blower, preventing it from starting. Solution: Turn off at the power supply and unplug the device. Remove the water chamber. Keep the device lid open and tip the device upside down to clear the water from the device. Place the water chamber back in the device. Restart the device. Is there an error message on the display? Solution: Refer to section <i>13.2 - Error Messages on SleepStyle Screen</i>, identify the error code and take the appropriate corrective action.
The pressure is fluctuating or insufficient air is being delivered from the device.	The patient's mask may not be fitted correctly, causing leaks. Solution: Ensure the mask is correctly fitted. Refer the patient to their mask's Use and Care Guide for fitting instructions. The device lid may not be closed correctly, the chamber seal may not be fitted to the water chamber correctly, or the outlet seal is missing. Solution: Ensure the chamber seal is correctly secured to the water chamber. Ensure the outlet seal and water chamber are in the device. Refer to section <i>10.3 - Reassembly of the Device and Accessories</i> for detailed instructions on reassembly of these parts. There may be water in the breathing tube. Solution: Disconnect the breathing tube and hang with both ends pointing to the floor until all water in the breathing tube has been cleared.
Auto Start / Stop is activated but the device does not start therapy automatically when the patient breathes into the mask.	The patient's breath might be too shallow to automatically start therapy when Auto Start / Stop is activated. Solution: Instruct the patient to take a few deep breaths with their mask on to automatically start therapy. Alternatively, the patient can press Start/Stop  to start therapy.

The base of the device is warm to the touch even though the device isn't being used.	This is normal and should not cause concern. In stand-by mode, the device consumes approximately 5 W of power. This may cause the feeling of warmth.
There is a build-up of water on the heater-plate.	When therapy has stopped, humidity in the water chamber can cool, causing condensation to form. Solution: To reduce condensation, the device should be plugged in and switched on at the power supply after stopping therapy. Before each use, the water chamber should be removed and the chamber housing dried with a cloth. If the water build-up becomes excessive, please contact your Fisher & Paykel Healthcare representative for further information.
The humidifier doesn't appear to be working.	The humidity level may be incorrect. Solution: Check the humidity level is above 0. See section 5 - <i>Comfort Options</i> for detailed information on the humidity setting.
	The water chamber may be empty. Solution: Check if there is water in the water chamber. Refer to section 8.1 - <i>Device Setup</i> for instructions on filling the water chamber.
	The ThermoSmart breathing tube is not connected correctly to the device. Solution: Remove the ThermoSmart breathing tube from the device and reconnect, making sure that the electrical connector on the breathing tube is correctly fitted into the device. When connected correctly, the ThermoSmart icon  appears on the device home screen. If the ThermoSmart icon  does not appear, the ThermoSmart breathing tube is not working and should be replaced.
Report not downloading from the InfoUSB or the InfoSmart report is missing data.	The device failed to detect the InfoUSB. Solution: Remove the InfoUSB and insert it fully into the InfoUSB port of the device. When inserted correctly, the InfoUSB icon  appears on the device home screen.
	The InfoUSB may have been removed whilst data was syncing, used to store other files, or may be full. Solution: Use a computer to delete the InfoUSB contents and reinsert for at least 2 minutes.
	The InfoUSB is faulty. Solution: Replace the InfoUSB. See section 12 - <i>Replacement Parts</i>.
The device will not pair to a mobile device using Bluetooth.	The device may not be discoverable or Bluetooth may not be turned on either device. Solution: Turning the Bluetooth setting off and on again on the mobile device may resolve connectivity issues. Refer to section 4.2 - <i>View your therapy data on the SleepStyle App or website</i> in the F&P SleepStyle+ Use and Care Guide for instructions on changing the Bluetooth setting.

13.2 Error Messages on SleepStyle Screen

If a fault is detected with the device or its accessories, a screen will appear with a notification and an error code. Identify the error code in the ranges specified below and follow the appropriate corrective action. If the error continues or reoccurs, record the number displayed and contact your Fisher & Paykel Healthcare representative for further instructions.

Error codes between	Solution
100–199	The device may not be able to provide effective therapy. The device may have shut down or may not be able to provide the prescribed pressure. Solution: Turn the power off and on at the power supply to restart the device.
400–499	Humidity may have been disabled. The device is still safe to use without humidity. The patient will still be treated at the prescribed pressure. Solution: Turn the power off and on at the power supply to restart the device.
510 or 512	There may be a problem with the ThermoSmart breathing tube. The device is still safe to use. The patient will still be treated and get humidity, but it may not be optimal. Solution: Try reconnecting your ThermoSmart breathing tube. When connected correctly, the ThermoSmart icon  will appear on the home screen. Alternatively, turn the power off and on at the power supply to restart the device. If the error continues or reoccurs, replace the ThermoSmart breathing tube. See section 12 - <i>Replacement Parts</i>.
500–599 (excluding 510 or 512)	The ThermoSmart breathing tube may have been disabled. The device is still safe to use. The patient will still be treated and get humidity, but it may not be optimal. Solution: Turn the power off and on at the power supply to restart the device. If the error continues or reoccurs, See section 12 - <i>Replacement Parts</i>.

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Tel: +34 902 013 346 **Sweden (SE)** Tel: +46 8 564 76 680

SleepStyle, SensAwake, F&P InfoUSB, ThermoSmart, and F&P InfoSmart are trademarks of Fisher & Paykel Healthcare Ltd.

The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG Inc. and any use of such marks by Fisher & Paykel Healthcare is under license. Other trademarks and trade names are those of their respective owners.

Android and Google Play are trademarks of Google Inc.

“Made for iPhone” means that an electronic accessory has been designed to connect specifically to iPhone, and has been certified by the developer to meet Apple performance standards. Apple is not responsible for the operation of this device or its compliance with safety and regulatory standards. Please note that the use of this accessory with iPhone may affect wireless performance. Apple and iPhone are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc., registered in the U.S. and other countries.

For patent information, see www.fphcare.com/ip

REF 621513 REV C 2022-08 © 2022 Fisher & Paykel Healthcare Limited

Fisher & Paykel
HEALTHCARE

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France (FR)  Fisher & Paykel Healthcare SAS, 10 Av. du Québec, Bât F5, BP 512, Villebon-sur-Yvette, 91946